# CV2S2000

<b>MANUALE DI UTILIZZO</b>	<b>5</b>
<b>USER MANUAL</b>	<b>11</b>
<b>MANUAL DE USO</b>	<b>17</b>
<b>NOTICE D'UTILISATION</b>	<b>23</b>
<b>BENUTZERHANDBUCH</b>	<b>29</b>

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---

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# *CV2S2000*

# *Applicatore*

# *professionale*

# *con coppia di*

# *solenoidi*

*I.A.C.E.R. Srl*

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---

MNPG366-00 Edition 25/08/2021

*CV2S2000*  
*Professional*  
*applicator with two*  
*solenoids*

I.A.C.E.R. Srl

[www.itechmedicaldivision.com](http://www.itechmedicaldivision.com)

### **Introduction**

The **CV2S2000** device is a professional two-solenoid applicator for low-frequency magnetotherapy. It is easily adaptable to any area of the body thanks to the presence of the therapeutic elastic tape.

The device is classified as class I, in accordance with rule 1 (4.1) referred to in Annex VIII of Regulation (EU) 2017/745 of the European Parliament and of the Council, dated 5 April 2017, relating to medical devices.

Basic UDI-DI: 8019781PEMFLFAPPP8


UDI-DI: (01)08019781472076

*Technical  
data*

### **Technical specifications:**

- *Applicator size: diameter 11 cm + elastic tape of 115 cm;*
- *Cable length: 184 cm;*
- *2 professional solenoids;*
- *1 connection cable;*
- *Compatible with MAG2000 and LaMagneto series models.*

*Labelling*


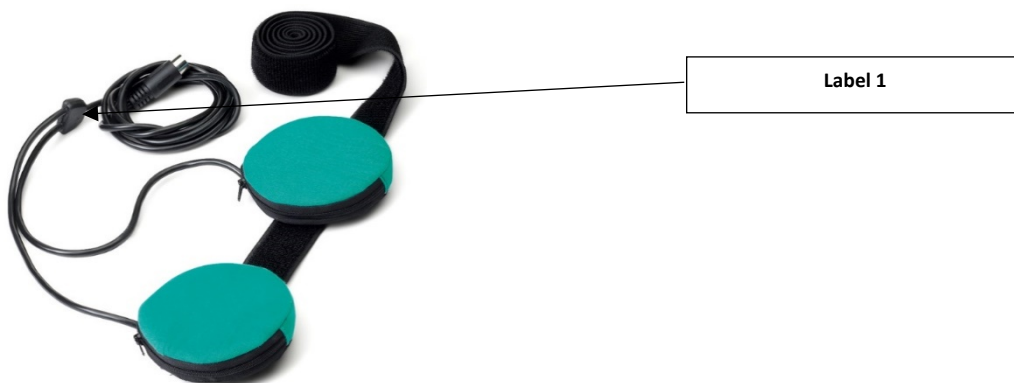
**Label 1**

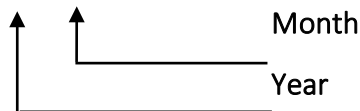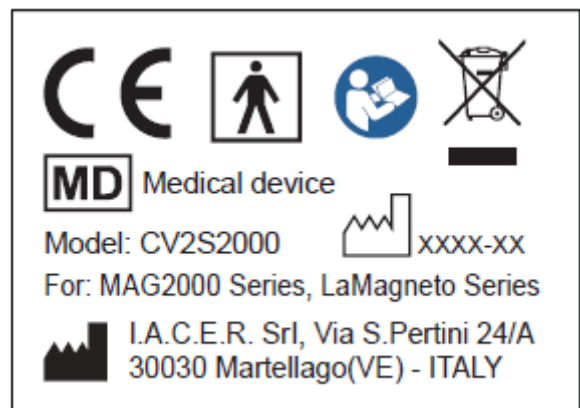29x18.7mm

**Label 2 - (Cable label)**

**Symbols:**

Symbol	Meaning
	Manufacturer's logo.
	In compliance with regulation (EU) no. 2017/745
	Manufacturer data.

Symbol	Meaning
	Date of manufacture (YYYYY-MM).
	Follow the instructions for use.
	Medical device.
	WEEE directive for the disposal of electronic waste.
	Type BF applied part.
	Allowed temperatures (storage temperatures, on packaging).
	Relative humidity (relative humidity of storage, on packaging).

## Transport and storage

*There are no special care instructions for transport. For storage, the equipment is protected up to the following environmental conditions:*

*room temperature from +5 to + 40°C*

*relative humidity from 15 to 93%*

## Instructions for use

### **Instructions for using CV2S2000**

*Follow the instructions in the User Manual supplied with the device in your possession.*

*Follow the Instructions given in the user manuals of the products of the MAG2000 or LaMagneto series.*

## How to take care of CV2S2000

### **Cleaning the CV2S2000**

To clean the CV2S2000 device, it is recommended to disconnect the professional two-solenoid applicator from the device before carrying out any operation. Clean the fabric with a cloth dampened with simple water and neutral soap and wait for it to dry completely before reusing the two-solenoid applicator for a therapy session.

### **Information for disposal**

The product is subject to the WEEE legislation (presence on the label of the symbol  ) relating to separate waste collection: to dispose of the product, use special areas equipped for the collection of electronic material by contacting the competent authorities of your country or the manufacturer directly.

### **Maintenance and Support**

The device does not require any particular routine maintenance. Please contact the manufacturer to provide for a check of the functionality of the device, every two years. The manufacturer is the sole entity authorized to provide technical support for the CV2S2000 applicator. Should you need technical support, please contact:

**I.A.C.E.R. S.r.l.**

Via S. Pertini, 24/a • 30030 Martellago (VE)

Ph. 041.5401356 • Fax 041.5402684

### **Warranty**

CV2S2000 is covered by a 2-year warranty starting from the date of purchase. The warranty shall lapse in case of tampering, neglect and inappropriate use of the applicator and in case of any intervention carried out by personnel who is not authorized by the manufacturer or by the authorized dealer.

All rights reserved. CV2S2000 and the logo  are the exclusive property of I.A.C.E.R. Srl and are registered trademarks

## MANUAL DE USO

---

MNPG367-00 Edición 25/08/21

# *CV2S2000*

# *Aplicador*

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# *par de*

# *solenoides*

*I.A.C.E.R. Srl*

*[www.itechmedicaldivision.com](http://www.itechmedicaldivision.com)*

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*Alle Rechte vorbehalten. CV2S2000 und das Logo  sind das exklusive Eigentum von I.A.C.E.R. Srl und sind registriert*