

I-TECH
MEDICAL DIVISION

**Pain relief in the
comfort of your
own home.**

You are in good hands

Trust the specialists in pain therapy and home rehabilitation.


We are here to guide you on your rehabilitation journey,
taking care of your health until your problem is solved.

What we do for you

We manufacture electromedical devices that are **safe and easy** to use on your own. Each of them comes with **pre-set therapeutic programs**, allowing you to start therapy in a few simple steps. And, should you need our support, **we are always ready to help you**.

In addition, our devices are **clinically validated and certified** by the notified bodies by the Ministry of Health, TÜV Rheinland Italia and MTIC Intercert.

We are ready for the telemedicine connection!

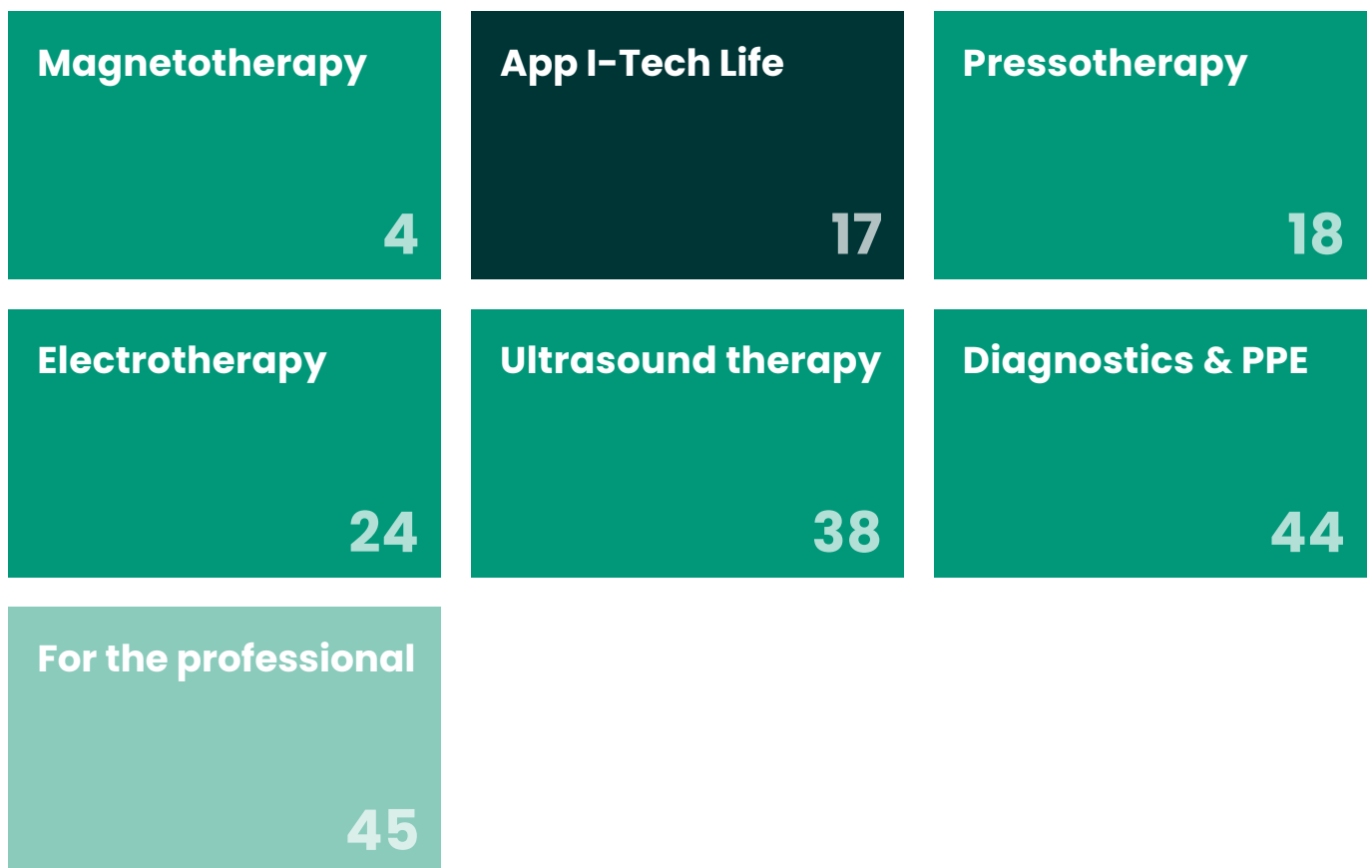

Consult your doctor. You will receive an accurate diagnosis, we will be able to guide you more easily towards the ideal solution for your needs.

THERAPEUTIC INDICATIONS

INFLAMMATORY PATHOLOGIES

OSTEOPOROSIS

ARTHROSIS

ARTHRITIS

INFLAMMATION

BONE OEDEMA

FRACTURES


LUMBAGO

ALGODYSTROPHY

JOINT PAIN

ULCERS

NEUROPATHIES

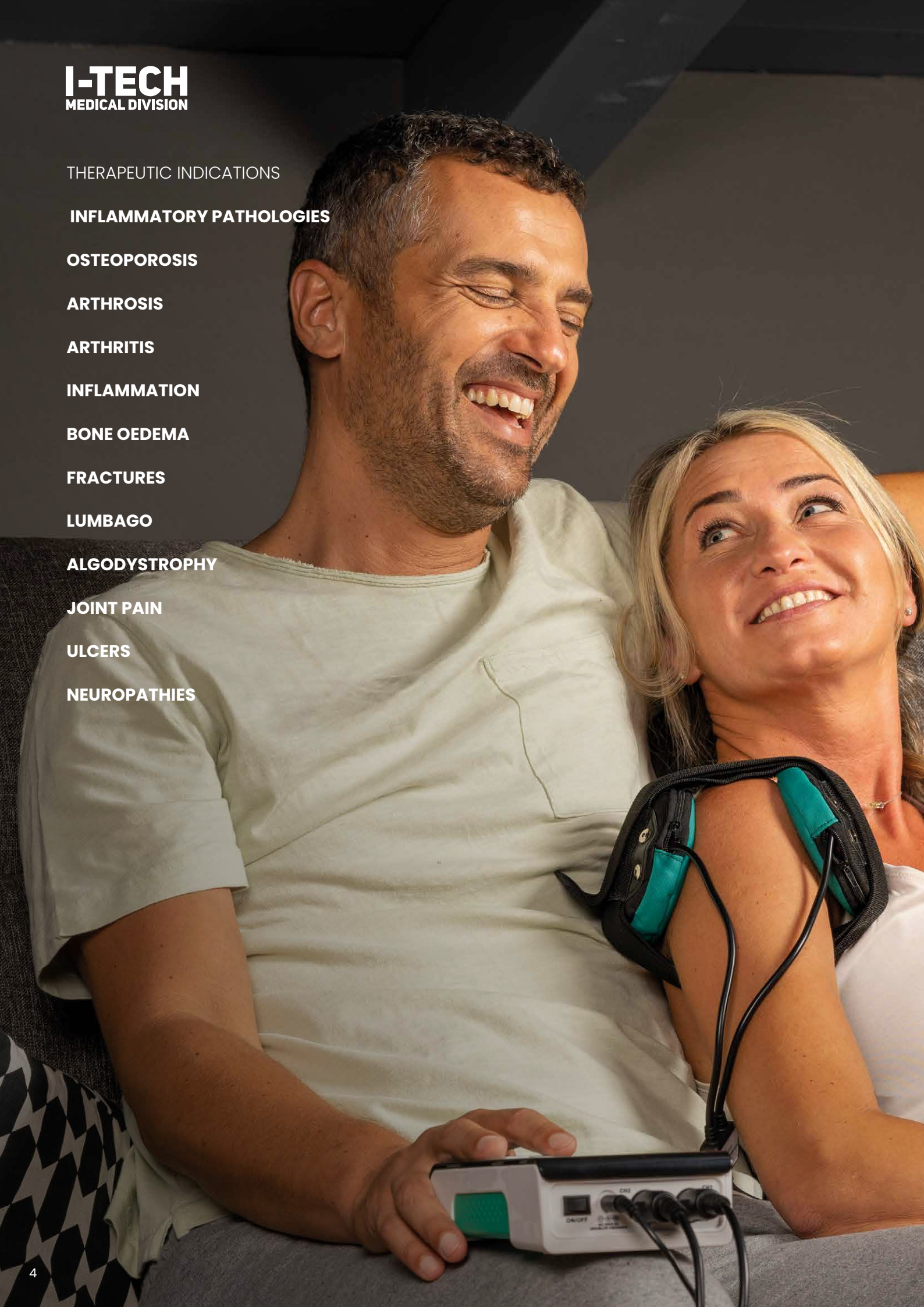

MAGNETOTHERAPY / PEMF

Reduce pain and inflammation with magnetotherapy. The ideal ally for treating traumatic and chronic conditions affecting bones, muscles and joints. The action of pulsed electromagnetic fields will help your body's cells restore their balance and proper functioning.

RECOMMENDED DURATION OF TREATMENT



At least 2-4 hours per day, depending on the pathology.



45-90 days. Repeat, if necessary.

LAMAGNETO

Reduce pain and inflammation from the comfort of your own home.

Doing magnetotherapy has never been so easy!



The first MDR-certified magnetotherapy device in Europe.



Recommended by doctors.

INTENSITY UP TO 150 GAUSS

for each of the 2 channels



Connect it to the APP



2 TREATMENTS PER SESSION

and get more results in less time

46 PRESET Programs

to make your therapy easy

FREQUENCY FROM 1 TO 100 HZ

ULCERS AND NEUROPATHIES

The only magnetotherapy device certified for both ulcers and neuropathies.

Included



Add Osteomat!

The mattress for long-lasting full-body treatments.

Find it out on page 12



PROGRAMS

01 Osteoporosis	13 Lumbosacral pain	25 Tendinitis	37 Impingement syndrome
02 Arthrosis	14 Shoulder arthrosis	26 Chondropathy	38 Carpal tunnel syndrome
03 Arthritis	15 Knee arthrosis	27 Anti-inflammatory	39 Titanium prosthesis
04 Cervical arthrosis	16 Scapulohumeral periarthrosis	28 Whiplash	40 Rotator cuff
05 Joint pain	17 Coxarthrosis	29 Scarring	41 Tarsal tunnel
06 Cervicalgia	18 Muscle atrophies	30 Skin ulcers	42 Night 1 Hz
07 Chronic pain	19 Muscle treatment	31 Discopathy	43 Night 15 Hz
08 Fractures	20 Osteonecrosis	32 Myalgia	44 Night 50 Hz
09 Epicondylitis	21 Algodystrophy	33 Neuropathy	45 Night 75 Hz
10 Epitrocleitis	22 Cartilage lesion	34 Muscle tear	46 Autoscan
11 Pseudoarthrosis	23 Ligament injury	35 Muscle cramp	
12 Lumbago	24 Bone oedema	36 Rhizarthrosis	



LAMAGNETO PRO

Even simpler, even more secure. Lock the program with a password and always start the correct therapy.



63 PRESET PROGRAMS

to make your therapy easy

CUSTOMISED TREATMENTS

thanks to 10 free memories

2 TREATMENTS PER SESSION

and get more results in less time

LOCK WITH PASSWORD

and you are sure you are not varying the correct therapy parameters



TRACK THE THERAPIES PERFORMED

with integrated counter

INTENSITY UP TO 200 GAUSS

for each of the 2 channels

FREQUENCY FROM 1 TO 120 HZ

to set precisely the parameters prescribed by your doctor



The 1st MDR-certified CEMP magnetotherapy device in Europe.



Connect it to the APP




Recommended by doctors.

Included




PROGRAMS (LAMAGNETO PRO AND LAMAGNETO X)

01 Osteoporosis	09 Epicondylitis	17 Coxarthrosis	25 Tendinitis
02 Arthrosis	10 Epitrocleitis	18 Muscle atrophies	26 Chondropathy
03 Arthritis	11 Pseudoarthrosis	19 Muscle treatment	27 Anti-inflammatory
04 Cervical arthrosis	12 Lumbago	20 Osteonecrosis	28 Whiplash
05 Joint pain	13 Lumbosacral pain	21 Algodystrophy	29 Scarring
06 Cervicalgia	14 Shoulder arthrosis	22 Cartilage lesion	30 Skin ulcers
07 Chronic pain	15 Knee arthrosis	23 Ligament injury	31 Discopathy
08 Fractures	16 Scapulohumeral periarthrosis	24 Bone oedema	32 Myalgia

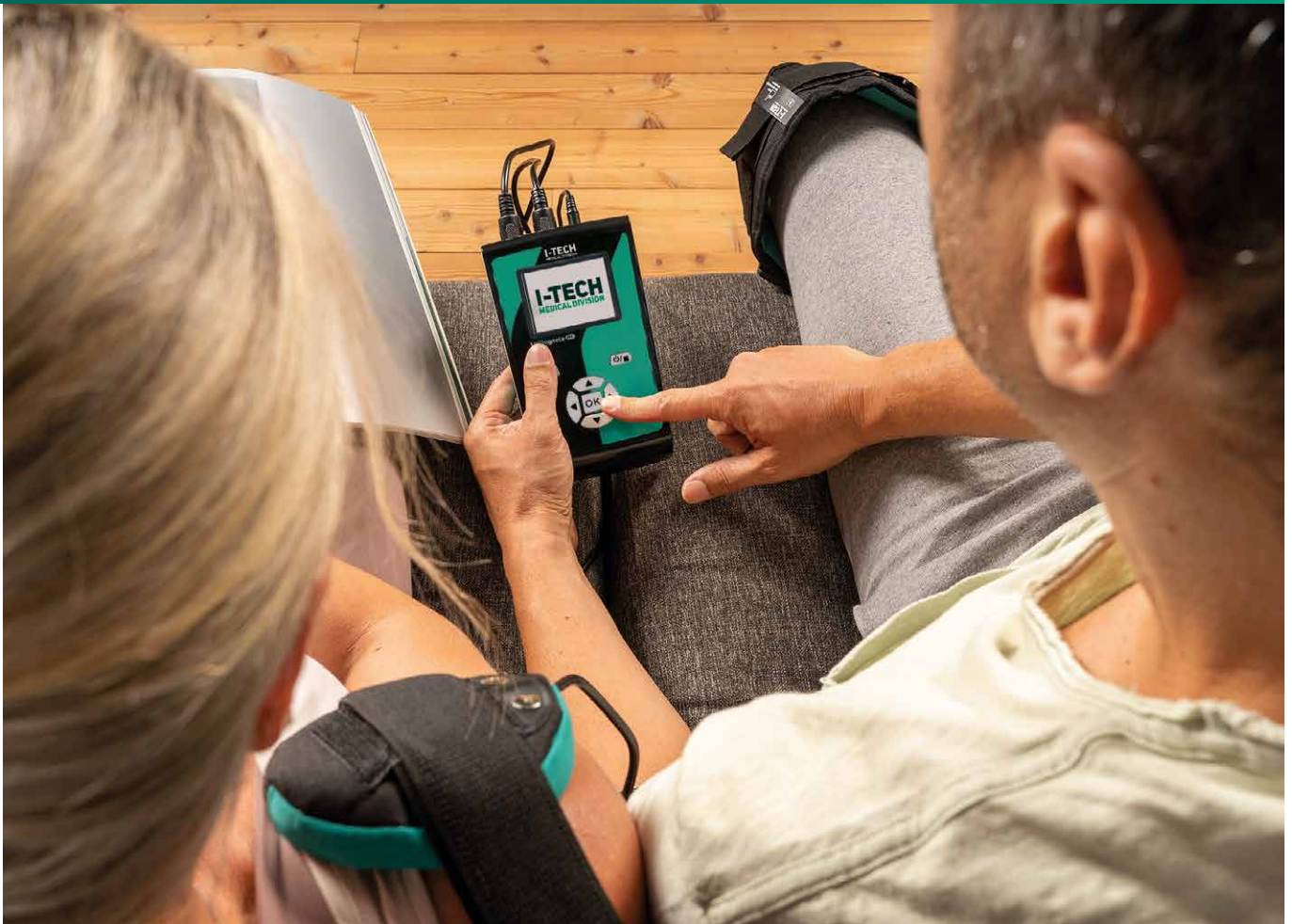

Switch to
LAMAGNETO X

LaMagneto Pro features plus Osteomat mattress.
For full-body therapy, even at night.

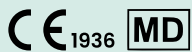

33 Neuropathy	41 Tarsal tunnel	49 5 Hz treatment	57 50 Hz treatment
34 Muscle tear	42 Night 1 Hz	50 8 Hz treatment	58 60 Hz treatment
35 Muscle cramp	43 Night 15 Hz	51 10 Hz treatment	59 70 Hz treatment
36 Rhizarthrosis	44 Night 50 Hz	52 15 Hz treatment	60 75 Hz Treatment
37 Impingement syndrome	45 Night 75 Hz	53 16 Hz treatment	61 80 Hz treatment
38 Carpal tunnel syndrome	46 Autoscan	54 20 Hz treatment	62 90 Hz treatment
39 Titanium prosthesis	47 1 Hz treatment	55 30 Hz treatment	63 100 Hz treatment
40 Rotator cuff	48 4 Hz treatment	56 40 Hz treatment	

MAG700

The 1-channel device for treating major osteoarticular pathologies.



CE 0068

11 programs
-
1 channel
Intensity up to 70 gauss
Frequency from 5 to 100 Hz

Included



F2S700

PROGRAMS (MAG700)

01 Osteoarthritis	07 Inflammatory
02 Arthritis	08 Autoscan
03 Osteoporosis	09 5 Hz treatment
04 Fractures	10 10 Hz treatment
05 Articular pain	11 20 Hz treatment
06 Muscle pain	

MAG2000

Since 15 years, the world's best-selling device. For you, in its original version.



CE 0068

35 preset therapy programs
-
2 channels
Intensity up to 150 gauss per channel
Frequency from 1 to 100 Hz

Included



F3S2000

PROGRAMS (MAG2000 - MAG2000 PREMIUM - MAG2000 PLUS)

01 Osteoporosis	07 Sprains
02 Arthrosis	08 Fractures
03 Arthritis	09 Epicondylitis
04 Cervical arthrosis	10 Epitrocleitis
05 Articular pain	11 Intercostal contusions
06 Cervicalgias	12 Lumbalgias

MAG2000 PREMIUM

Two bands already included. To treat your pathology on several parts of the body.



35 preset therapy programs
-
2 channels
Intensity up to 150 gauss per channel
Frequency from 1 to 100 Hz

MAG2000 PLUS

Are you wearing plaster? Mag2000 Plus is also ideal for high-intensity treatments.



35 preset therapy programs
+ 10 customisable free memories
2 channels
Intensity up to 200 gauss per channel
Frequency from 1 to 120 Hz

Included



F3S2000

Included



F3S2000

CV2S2000

13 Lumbar pain	19 Muscle contractures	25 15 Hz treatment	31 70 Hz treatment
14 Shoulder arthrosis	20 Osteonecrosis	26 20 Hz treatment	32 80 Hz treatment
15 Knee arthrosis	21 1 Hz treatment	27 30 Hz treatment	33 90 Hz treatment
16 Periarthritis	22 3 Hz treatment	28 40 Hz treatment	34 100 Hz treatment
17 Coxarthrosis	23 5 Hz treatment	29 50 Hz treatment	35 Autoscan
18 Athropy	24 10 Hz treatment	30 60 Hz treatment	

CUSTOMISE YOUR THERAPY

Complete your device with accessories that fit your needs.

Diffuse pain?

OSTEOMAT

Soft, light, foldable. The mattress for long-lasting, full-body care. Even while you sleep.

Compatible with LaMagneto, LaMagneto Pro, LaMagneto X, Mag2000, Mag2000 Premium, Mag2000 Plus

TREATS THE WHOLE BODY

thanks to 8 solenoids positioned at shoulder, spine, pelvis and knee height

EXTREMELY SOFT

for therapy at night, while you sleep

LIGHTWEIGHT AND FOLDABLE

to carry it wherever you want



Articular pain?

CV2S2000 SOLENOID PAIR

Maximum comfort when treating your joints, even at high intensity.

Compatible with LaMagneto, LaMagneto Pro, LaMagneto X,
Mag2000, Mag2000 Premium, Mag2000 Plus



F3S2000 THERAPEUTIC BELT

Therapeutic elastic belt for low-frequency magnetotherapy with 3-solenoid cable.

Compatible with LaMagneto series, Mag2000, Mag2000 Premium, Mag2000 Plus.



THERAPEUTIC BELT F2S700

Elastic therapeutic belt for low-frequency magnetotherapy with 2-solenoid cable.

Compatible with Mag700.



TAP2000

The 40x40cm mat with 6 solenoids, ideal for your back treatments.

Compatible with LaMagneto series, Mag2000, Mag2000 Premium, Mag2000 Plus.



CIL-MAG

The cylindrical professional solenoid which allows you to comfortably treat the lower limbs.

Compatible con Mag2000, Mag2000 Premium, Mag2000 Plus.



ORTHOMAG

Are you always on the move?
With Orthomag you have therapy wherever you want, in total freedom.

PORTABLE, WITH RECHARGEABLE BATTERY

for therapy wherever you want, whenever you want



INTENSITY
UP TO 25 GAUSS

FLEXIBLE APPLICATOR



CE 0068

Included



TREATMENTS

PROGRAM L (50 HZ)

- Algodystrophies
- Gonarthrosis
- Cartilage degeneration
- Fractures
- Cartilage injuries
- Osteoarthrosis
- Osteonecrosis
- Osteoporosis
- Delayed calcification
- Pseudo-arthritis

PROGRAM H (75 HZ)

- | | |
|---------------------|-----------------------|
| Scar adherence | Epicondylitis |
| Arthritis | Epitrochleitis |
| Arthrosis | Discal Hernia |
| Bursitis | Plantar fasciitis |
| Brachialgy | Lumbago |
| Capsulitis | Meniscopathy |
| Cervical pain | Metatarsal pain |
| Whiplash | Periarthritis |
| Chondropathy | Pubalgy |
| Contusions | Rheumatisms |
| Coxarthrosis | Rhizarthrosis |
| Articular pain | Sciatalgy |
| Rheumatic disorders | Inflammatory diseases |
| Back pain | Muscle strains |

MAG1000

The high-frequency magnetotherapy device to treat mainly muscle or tendon pathologies.

20 PRESET PROGRAMS
to make your therapy super simple

2 CHANNELS

3 LEVELS OF INTENSITY
adjustable



FREQUENCY
27 MHz

MODULATION
8 TO 640 Hz




Add TAP-HF!
The 40x40cm mat for high-frequency magnetotherapy




PROGRAMS

01 Arthritis / Arthrosis	06 Osteoporosis	11 Bruises	16 Epytrocleitis
02 Rheumatisms	07 Fractures	12 Myalgia	17 Lumbago
03 Articular prosthesis	08 Sprains	13 Tendonitis	18 Muscular contractures
04 Herniated disc	09 Dislocations	14 Hematoma	19 2h Medium application
05 Muscular pain	10 Traumas	15 Epicondyliti	20 4h Long application

Get the most out of your therapy.

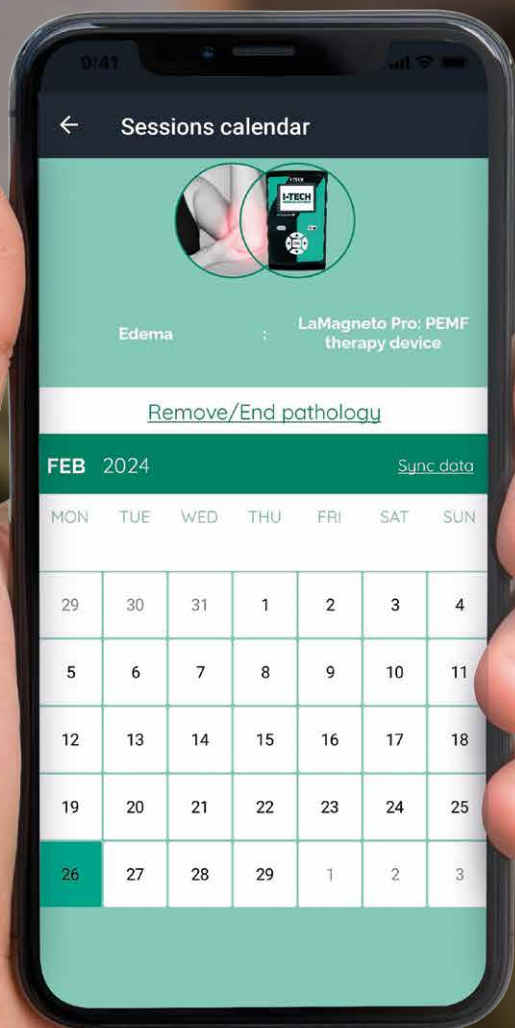

**PLAN
YOUR SESSIONS**

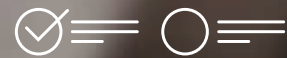

**RECORD YOUR
PROGRESS**



**ACCESS EXCLUSIVE
CONTENT**

Download our
app and start
your therapeutic
journey.

I-TECH
Life

Download on the
App Store

GET IT ON
Google Play

THERAPEUTIC INDICATIONS

EDEMA

LYMPHEDEMA

VENOUS ULCERS

VENOUS INSUFFICIENCY

MUSCLE RECOVERY



PRESSOTHERAPY

Swollen or stiff legs?

Pressotherapy will help you reduce swelling, limb stiffness and remove toxins.

It also promotes proper venous and lymphatic return in arms and legs.

RECOMMENDED TREATMENT DURATION



From 20 to 60
minutes per day



30 days

HOW DOES IT WORK?

Pressotherapy exerts sequential compression on the limbs and abdomen with a disto-proximal movement, i.e. from the periphery towards the centre of the body. This action favours the movement of blood and lymph in the presence of stagnation or circulation difficulties.

I-PRESS

I-Press helps you improve venous and lymphatic circulation of the limbs and treat certain pathologies of the circulatory system.



Clinical studies on edema and recovery post-arthroplasty with our device



Recommended by doctors.

4 COMPRESSING CHAMBERS

which inflate in sequence from foot to thigh

CHOOSE CONFIGURATION AND ACCESSORIES

customise the treatment as you wish

EASY AND INTUITIVE

set session duration and pressure and you are ready for the therapy



CONFIGURATIONS

I-Press LEG1

- Machine body
- 1 legging
- 4-terminal tube
- 1 presser foot insole



I-Press LEG2

- Machine body
- 2 leggings
- 8-terminals tube
- 2 pressure insoles



I-Press ARMI

- Machine body
- 1 cuff (right or left)
- 4-terminal tube



All 3 configurations can be further customised with the full range of pressotherapy accessories.

TREATMENTS

Pressure (mmHg)

Time (minutes)

TREATMENTS	Pressure (mmHg)	Time (minutes)
Edema	50	50
Lymphedema	50	30
Venous ulcers	50	60
Venous insufficiency	50	60
Muscle recovery	85	20

LEGGINGS

4-chambers

LEGGING

THIGH CIRCUMFERENCE

ANKLE CIRCUMFERENCE

LEG LENGTH

(measurement taken from inner thigh to heel)

EXTENSORS

Pair of leg extenders

TNT PROTECTION

Pair of hygienic TNT protectors for leggings

SIZE (M)



Up to 60 cm

Up to 33 cm

Up to 76 cm

SIZE (L)



Up to 72 cm

Up to 46 cm

Up to 84 cm



ARMBAND

4-chambers

ARMBAND

Right and left

ARMPIT CIRCUMFERENCE

WRIST CIRCUMFERENCE

ARM LENGTH

(measurement taken from armpit to wrist)

EXTENSORS

Armband extender

ARMBAND


(armband + extensor)

CIRCUMFERENCE WRIST

(armband + extender)

ARM LENGTH

(measurement taken from armpit to wrist)

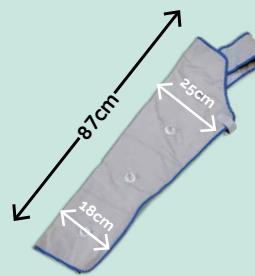

Up to 46 cm

Up to 32 cm

Up to 87 cm



Up to 59 cm

Up to 37 cm


Up to 87 cm

ABDOMINAL BAND

4-chambers abdominal belt


WAIST CIRCUMFERENCE

SIZE **(M)**

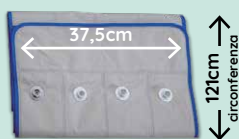

Up to 116 cm

SIZE **(L)**

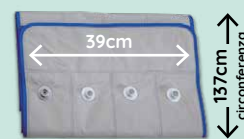

From 117 cm to 132 cm

The photos of the accessories are purely illustrative, the product may show variations in some aesthetic details.

ACCESSORIES

DOUBLE CONNECTOR

Connector with 4 tubes and 8 terminations for simultaneous therapy (2 leggings or 2 armbands)



SINGLE CONNECTOR

4-tube-connector for armband, legging or abdominal band.



DUAL THERAPY ADAPTER

Adapter for simultaneous therapy (2 leggings and abdominal band).



The photos of the accessories are purely illustrative, the product may show variations in some aesthetic details.

THERAPEUTIC INDICATIONS

MUSCLE RECOVERY

PAIN REDUCTION

INCONTINENCE TREATMENT

**TREATMENT OF
DENERVATED MUSCLE**

FLUID DRAINAGE



ELECTROTHERAPY

Recover faster, reducing pain.

Electrotherapy transmits micro-impulses to the human body, varying in frequency pulse width and intensity, depending on the treatment to carry out.

RECOMMENDED TREATMENT DURATION



To find out the duration of therapy we recommend that you consult your doctor and follow the instructions in our user manual.

MIO-CARE TENS

Reduce pain quickly.
In case of both and acute pain.



CE 0068

20 TENS programs
-
-
2 independent channels
Compensated biphasic square wave
Belt clip
Maximum intensity: 100mA
Internal rechargeable Ni-Mh battery

MIO-CARE FITNESS


Find pain relief, reduce blemishes and strengthen your muscles. All with Mio-Care Fitness.




CE 0068

20 TENS programs
20 FITNESS programs
15 BEAUTY programmes
2 independent channels
Compensated biphasic square wave
Belt clip
Maximum intensity: 100mA
Internal rechargeable Ni-Mh battery

Included

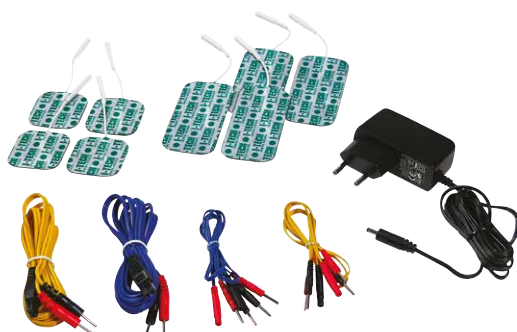

Included

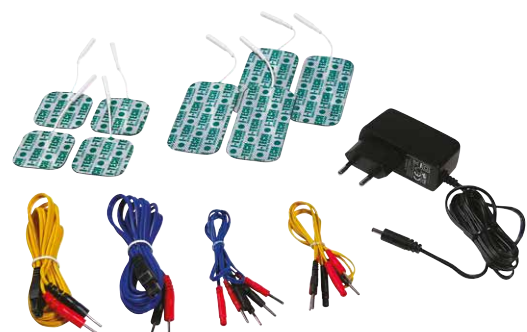

PROGRAMS

TENS

(MIO-CARE TENS E MIO-CARE FITNESS)

01 Fast TENS
02 Endorphinic tens (delayed)
03 Tens at maximum values
04 Anti-inflammatory
05 Neck pain / Headache
06 Back / Sciatic pain
07 Sprains / Bruises
08 Vascularisation
09 Muscle relaxant
10 Haematomas
11 Atrophy prevention
12 Atrophy (trophism rehabilitation)
13 Hand and wrist pain
14 Plantar stimulation
15 Epicondylitis
16 Epitrochlea
17 Periarthritis
18 Microcurrent
19 Stress incontinence*
20 Urgency incontinence*

*Probes not included

FITNESS

(MIO-CARE FITNESS)

01 Warming up
02 Resistance upper limbs and trunk
03 Resistance lower limbs
04 Resistant strength upper limbs and trunk
05 Resistant strength lower limbs
06 Basic strength upper limbs and trunk
07 Basic strength lower limbs
08 Fast strength upper limbs and trunk
09 Fast strength lower limbs
10 Explosive strength upper limbs and trunk
11 Explosive strength lower limbs
12 Deep capillarisation
13 Muscle recovery
14 Agonist-antagonist
15 Sequential tonic contractions upper limbs and trunk
16 Sequential tonic contractions lower limbs
17 Sequential phasic contractions upper limbs and trunk
18 Sequential phasic contractions lower limbs
19 Muscle relaxant
20 Deep massage

BEAUTY

(MIO-CARE FITNESS)

01 Firming up upper limbs and trunk
02 Firming up lower limbs
03 Toning up upper limbs and trunk
04 Toning up lower limbs
05 Definition upper limbs and trunk
06 Definition lower limbs
07 Modelling
08 Microlifting
09 Lipolysis abdomen
10 Lipolysis thighs
11 Lipolysis glutei and hips
12 Lipolysis arms
13 Tissue elasticity
14 Capillarisation
15 Heaviness in legs

Want the top?

Switch to I-TECH PHYSIO

The easy-to-use and fully programmable electrostimulator for pain therapy and muscle rehabilitation

Find out more on page 30



MIO-CARE PRO

Strengthen your muscles, after an operation or to prepare for competition

2 INDEPENDENT CHANNELS

easily programmable

2 SQUARE WAVE TYPES

compensated biphasic and monophasic

BELT CLIP


for greater convenience



MAXIMUM INTENSITY
100 mA

INTERNAL BATTERY
Rechargeable Ni-Mh

Included

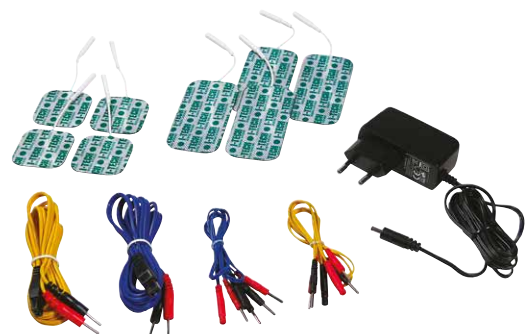



PROGRAMS

TENS

- 01 Conventional tens (fast)
- 02 Endorphinic tens
- 03 Tens at maximum values
- 04 Anti-inflammatory
- 05 Neck pain / Headache
- 06 Backache/Sciatic pain
- 07 Sprains / Bruises
- 08 Vascularisation
- 09 Muscle relaxant
- 10 Hand and wrist pain
- 11 Plantar stimulation
- 12 Epicondylitis
- 13 Epitroclea
- 14 Periarthritis

NEMS

- 01 Warming up
- 02 Resistance upper limbs and trunk
- 03 Resistance lower limbs
- 04 Resistant strength upper limbs and trunk
- 05 Resistant strenght lower limbs
- 06 Basic strength upper limbs and trunk
- 07 Basic strength lower limbs
- 08 Fast strength upper limbs and trunk
- 09 Fast strength lower limbs
- 10 Explosive strength upper limbs and trunk
- 11 Explosive strength lower limbs
- 12 Deep capillarisation
- 13 Muscle recovery
- 14 Agonist-antagonist
- 15 Sequential tonic contractions upper limbs and trunk
- 16 Sequential tonic contractions lower limbs
- 17 Sequential phasic contractions upper limbs and trunk
- 18 Sequential phasic contractions lower limbs
- 19 Muscle relaxant
- 20 Deep massage
- 21 EMS rehabilitation

BEAUTY

- 01 Firming up upper limbs and trunk
- 02 Firming up lower limbs
- 03 Toning up upper limbs and trunk
- 04 Toning up lower limbs
- 05 Definition upper limbs and trunk
- 06 Definition lower limbs
- 07 Modelling
- 08 Microlifting
- 09 Lipolysis abdomen
- 10 Lipolysis thighs
- 11 Lipolysis glutei and hips
- 12 Lipolysis arms
- 13 Tissue elasticity
- 14 Capillarisation
- 15 Heaviness in legs

FREE MEMORIES

- 01 TENS Mem1
- 02 TENS Mem2
- 03 TENS Mem3
- 04 TENS Mem4
- 05 TENS Mem5
- 06 NEMS Mem1
- 07 NEMS Mem2
- 08 NEMS Mem3
- 09 NEMS Mem4
- 10 NEMS Mem5
- 11 NEMS alternating
- 12 NEMS alternating
- 13 Battery test

I-TECH PHYSIO

The complete and programmable device to treat pain and rehabilitate your muscles.



Recommended
by doctors.

PROGRAMS

TENS

- 01 Rapid tens
- 02 Endorphinic tens
- 03 Tens at maximum values
- 04 Anti-inflammatory
- 05 Cervicalgia/Myotensive headache
- 06 Lumbago/Sciatica
- 07 Sprains/Contusions
- 08 Vascularisation
- 09 Decontracting
- 10 Hand and wrist pain
- 11 Plantar stimulation
- 12 Epicondylitis
- 13 Epitrocleitis
- 14 Periarthritis

REHA

- 01 Iontophoresis L
- 02 Iontophoresis M
- 03 Iontophoresis H
- 04 Microcurrent
- 05 Haematoma
- 06 Edema
- 07 Modulated Tens
- 08 Burst Tens
- 09 Atrophy prevention
- 10 Atrophy
- 11 Denervated quadra 50 ms
- 12 Denervated quadra 100 ms
- 13 Denervated quadra 150 ms
- 14 Denervated quadra 200 ms
- 15 Denervated quadra 250 ms
- 16 Denervated triangular 50 ms
- 17 Denervated triangular 100 ms
- 18 Denervated triangular 150 ms
- 19 Denervated triangular 200 ms
- 20 Denervated triangular 250 ms
- 21 Interferential
- 22 Tens with amplitude modulation
- 23 Alternating Tens
- 24 Tens with frequency modulation 1
- 25 Tens with frequency modulation 2
- 26 Tens with amplitude modulation
- 27 Kotz

2 INDEPENDENT CHANNELS

easily programmable

DENERVATED MUSCLE, IONTOPHORESIS

INTERFERENTIAL AND KOTZ WAVE

in addition to the classic compensated biphasic square, monophasic square and triangular



Connect it to the APP

BELT CLIP
for greater convenience

MAXIMUM INTENSITY 100 mA

INTERNAL BATTERY
Rechargeable Ni-Mh



Included

Possibility to add

Anal probe
Vaginal probe

NEMS

01 Warm-up	16 Tonic contractions sequential lower limb contractions
02 Upper limb resistance and trunk	17 Phasic contractions sequential upper limbs and trunk
03 Lower limb endurance	18 Phasic contractions sequential lower limb contractions
04 Resistance upper limbs and trunk	19 Decontracting
05 Lower limb strength	20 Deep tissue massage
06 Basic strength upper limbs and trunk	21 EMS
07 Basic strength lower limbs	
08 Fast strength upper limbs and trunk	
09 Fast strength lower limbs	
10 Explosive strength upper limbs and trunk	
11 Explosive strength lower limbs	
12 Deep capillarisation	
13 Muscle recovery	
14 Agonist-antagonist	
15 Sequential tonic contractions Upper limbs and trunk	

BEAUTY

01 Firming upper limbs and trunk
02 Lower limb toning
03 Upper limb toning and trunk
04 Lower limb toning
05 Upper limb definition and trunk
06 Lower limb definition
07 Modelling
08 Microlifting
09 Abdomen lipolysis
10 Lipolysis thighs
11 Lipolysis buttocks and hips
12 Lipolysis arms
13 Tissue elasticity
14 Capillarisation

INCONTINENCE

01 Stress incontinence*
02 Stress incontinence*
03 Stress incontinence*
04 Urge incontinence*
05 Urge incontinence*
06 Urge incontinence*
07 Mixed incontinence*
08 Mixed incontinence*
09 Mixed incontinence*

*Probes not included

+ 13 FREE MEMORIES





I-TECH
Life

Connect it to
the **APP**

DISCOVER THE T-ONE RANGE
T-One is our range of
4-channel electrostimulators.



T-ONE COACH

Tone and strengthen your muscles, find relief from pain and recover from traumas and injuries.



CE 0068

-
16 TENS programs
14 NEMS programs
18 BEAUTY programs
-
10 programs dual user (2+2)
-
-
4 independent channels (up to 8 electrodes)
Adjustable intensity from 1 to 120 mA
Wave frequency from 1 to 150 Hz
Pulse width from 20 to 300 µs
Preset time for each program
Compensated biphasic square wave
-
-
-
-

T-ONE MEDI SPORT

It improves your performance: before and after training, or to recover from an injury.



CE 0068

10 programs REHA
16 programs TENS
20 programs NEMS
18 programs BEAUTY
-
22 dual-user programs (2+2)
12 free TENS/NEMS memories
Includes Iontophoresis treatments
4 independent channels (up to 8 electrodes)
Adjustable intensity from 1 to 120 mA
Wave frequency from 1 to 1200 Hz
Pulse width from 20 to 450 µs
Time from 1 to 90 minutes
Compensated biphasic square wave
Monophasic wave (only for iontophoresis programs)
-
-
-

All T-One models have an internal rechargeable Ni-Mh battery.

Included



Included



T-ONE MEDI PRO

The ideal support for you who are professional athlete or work with sports teams.

Device used in clinical study in cooperation with the German Space Agency.



CE 0068

16 programs REHA
16 programs TENS
20 programs NEMS
18 programs BEAUTY
9 programs URO
22 dual-user programs (2+2)
12 free TENS/NEMS memories
Includes treatments for denervated muscle and iontophoresis
4 independent channels (up to 8 electrodes)
Adjustable intensity from 1 to 120 mA
Wave frequency from 0.2 to 1200 Hz
Pulse width from 20 µs to 300 ms
Time from 1 to 90 minutes
Compensated biphasic square wave
Monophasic wave (only for iontophoresis programs)
-
-
-

T-ONE REHAB

The most comprehensive device for you as a sport and rehabilitation specialist.

Post-arthroplasty clinical trials and Covid patients.



CE 0068

29 REHA programs
16 programs TENS
20 programs NEMS
18 programs BEAUTY
9 URO programs
22 Dual-user programs (2+2)
12 TENS/NEMS free memories
Includes treatments for denervated muscle and iontophoresis
4 independent channels (up to 8 electrodes)
Intensity adjustable from 1 to 120 mA
Wave frequency from 0.2 to 1200 Hz
Pulse width from 20 µs to 300 ms
Time from 1 to 90 minutes
Compensated biphasic square wave
Monophasic wave (only for iontophoresis programs)
Triangular wave
Interferential wave
Kotz wave

All T-One models have an internal rechargeable Ni-Mh battery.

Included




Included



ELJ41X414 / ELJ48X484 MD


4 adhesive pre-gelled electrodes with 2mm jack.
Measurements: 41x41mm
48x48mm

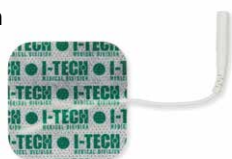

CE

ELJ40X804 / ELJ50X904 MD

4 adhesive pre-gelled electrodes with 2mm jack.
Sizes: 40x80mm
50x90mm

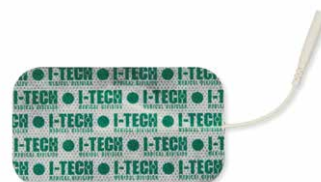

CE

OVAL03 MD

2 adhesive pre-gelled 130x76mm electrodes with 2mm jack



CE

ELJ30R4 MD

4 pre-gelled adhesive 30 mm face electrodes with 2mm jack



CE

ELB46X474 MD

4 pre-gelled adhesive 46x47mm electrodes with button attachment.



CE

ELB53X964 MD

4 pre-gelled adhesive 53x96mm electrodes with button attachment.



CE

ELADCL8F

4 cable adapters for 2 mm male plug electrode/plug with button attachment.



FP235-245
FC250-265 MD

4 conductive elastic bands for thighs 2x65cm +2x50 cm and band for calf 2x35 adjustable with button attachment.



CE

F180X60 MD

blu elastic 8x60 cm band for iontophoresis



CE

ELB60X801 / ELB80X1201 MD

Iontophoresis electrode made of buckskin with button attachment.
Sizes: 60x80mm
80x120mm



CE

ELG60X852/ELG80X1202 MD

2 rubber electrodes with 2 mm jack connection.
Dimensions: 60x85mm / 80x120mm



CE

The photos of the accessories are purely illustrative, the product may show variations in some aesthetic details.

SPX60X852/SPX80X1202 MD

2 sponges for rubber electrodes with 2 mm jack connection.
 Sizes: 60x85mm / 80x120mm



CE

IPP-PAD MD


Iontophoresis electrode specific for IPP (Induratio Penis Plastica) treatment.



CE

SA2S/BEAC

Beac anal probe
 2 mm jack connection

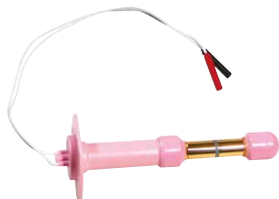

CE 0051

Device compliant with Directive 93/42/EEC

SV2S/BEAC

Vaginal probe
 Beac with 2 mm jack connection.



CE 0051

Device compliant with Directive 93/42/EEC

GE000250 MD

260ml gel for ultrasound therapy and electrotherapy.



CE

HANDY TRODE

bipolar handpiece/
 Electrode for face/neckline/
 breast with 2mm grip.



PEN-EL

Point searching pen with 2mm grip.



CV025318

Blue and yellow cable for Mio-Care series with terminal 2 mm terminal.



TWINJ2X2

2 yellow and 2 blue splitter cables with 2 mm terminal.



MIO-BATTERY NEW

Rechargeable battery Mio-Care and Mag1000 series (2012).



UNICHAR

Battery charger Mio-Care Tens/Beauty/Fitness and Mag1000 (2012).



UNICHAR NEW

Battery charger (6.8Vdc) T-One series, Mag1000 (2018), Mio-Vein, Mio-Care Pro, Mio-Ionotens and I-Tech Physio.



T-ONE BATTERY

Rechargeable T-One 2016 series 4.8V Ni-Mh 2300mA battery



CJ025317 / CJ025314 / CJ025312 / CJ025315

Blue, yellow, red, white T-One series cable with 2mm terminal.
 Sold individually.



The photos of the accessories are purely illustrative, the product may show variations in some aesthetic details.

THERAPEUTIC INDICATIONS

MYOFASCIAL PAIN

MUSCLE PAIN

CARPAL TUNNEL

VENOUS ULCERS

PRESSURE ULCERS

LUMBAR PAIN

ARTHROSIS

EPICONDYLITIS


EPITROCHLEITIS

TENDINITIS

CUBITAL TUNNEL

LUMBAR STENOSIS

SCIATICA

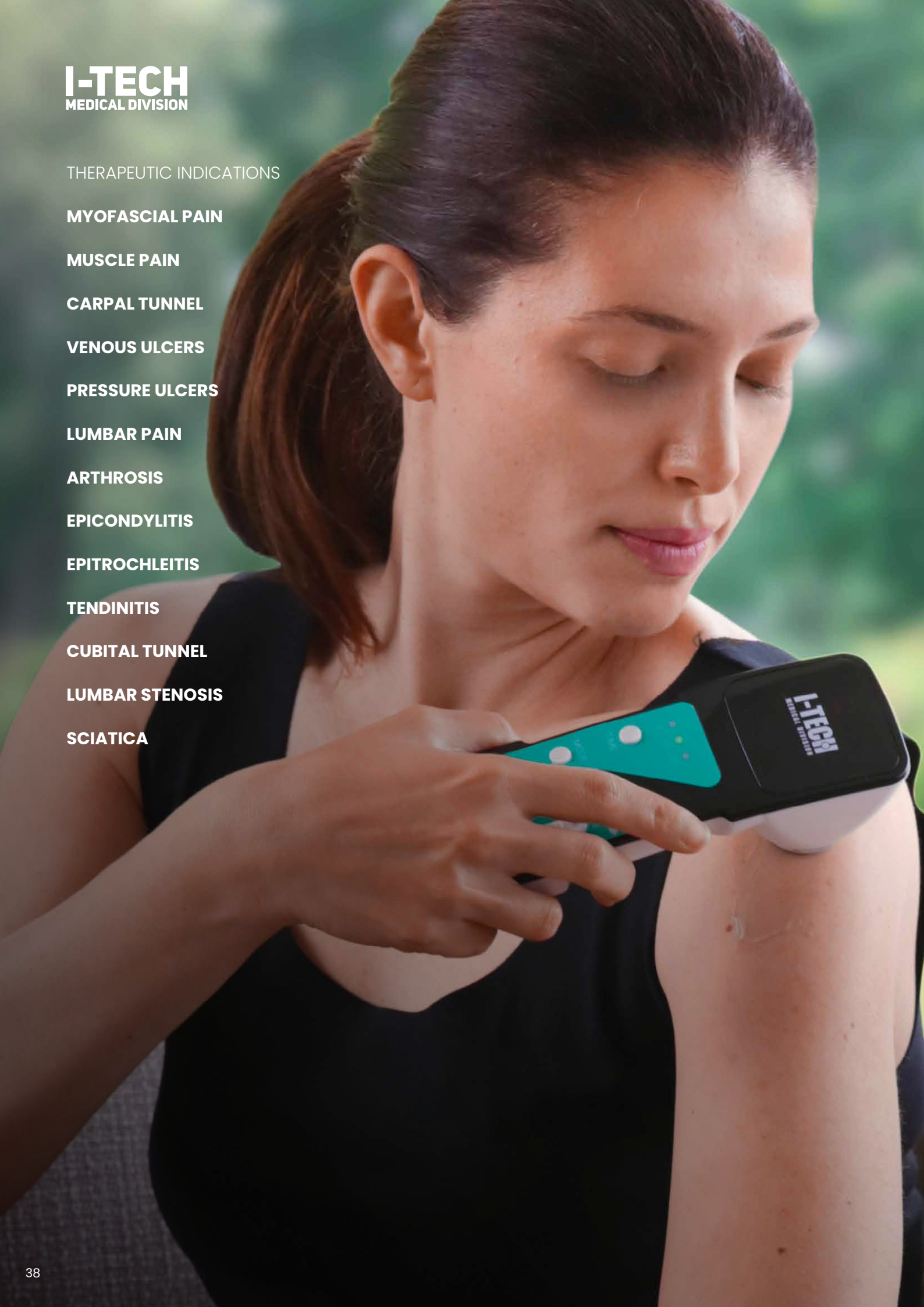

ULTRASOUND THERAPY

With ultrasound you can find relief from pain in just a few sessions.

By transferring energy to the tissues of our body, they promote microcirculation and help eliminate excess subcutaneous fluid properly.

RECOMMENDED TREATMENT DURATION



5 to 15 minutes
per day.



10 days.
Potentially
repeatable.

POWERSONIC

Do therapy where you want, when you want.
And reduce pain and inflammation
in just a few sessions.



Recommended
by doctors.



MAXIMUM INTENSITY
1.6 W/CM²
plugged into the socket

THE ONLY BATTERY-POWERED
among ultrasound
medical devices **CE**

1 MHZ ULTRASOUND
like a professional treatment

ADJUSTABLE DURATION AND INTENSITY
for 9 different combinations



TREATMENTS

Myofascial pain	Pressure ulcers	Epitrocleitis	Sciatica
Muscle pain	Lumbar pain	Tendinitis	
Carpal tunnel	Arthrosis	Cubital tunnel	
Venous ulcers	Epicondylitis	Lumbar stenosis	

MIO-SONIC

Reduce pain and contractures with the compactness of Mio-Sonic.
Also effective in the treatment of blemishes.

**COLLIMATED
ULTRASONIC BEAM**

**MAXIMUM INTENSITY
1.6 W/CM²**

1 MH ULTRASOUND

**ADJUSTABLE
DURATION AND
INTENSITY**

for 9 different combinations



PROGRAMS

9 combinations of duration and intensity, which can be set to perform 20 programs for analgic and decontracting treatments and 29 programs for beauty treatments.



Learn more about ultrasound therapy



Get the guide here

<https://itechmedicaldivision.com/en/ultrasound-therapy-what-is-it-for/>

DIAGNOSTICS

Monitor your vital parameters with pulse oximeters and the digital thermometer by I-Tech Medical Division.

FOX-200 Saturimeter

- LED display
- SpO2 indication, heart rate, pulse
- Auto power off when finger is not inserted
- Low battery indication
- Fits finger size of adults and children

NO NEWBORNS



CE 0482

Saturimeter FOX-300

- 2-colour OLED display
- SpO2 display, heart rate, pulse bar, perfusion index, plethysmogram
- Auto power off when finger is not inserted
- Low battery indication
- Fits finger size of adults and children

NO NEWBORNS



CE 0123

FI06 digital infrared thermometer

- 3-colour backlit LCD display
- Fever alarm indicator
- Instant reading
- Accurate measurement
- Front measurement in degrees °C/°F
- Available memory



CE 0197

PROFESSIONAL PRODUCTS

I-Tech Medical Division also in your practice!
Discover our devices for ambulatory use
for health professionals.

Diathermy

Diathermy, better known as Tecar therapy (acronym for Capacitive Resistive Energy transfer), is positioned as one of the most powerful therapeutic techniques currently on the market. Diathermy, in fact, transfers to the human body the largest amount of energy per unit of time. By generating heat directly inside the tissues, it accelerates metabolic and recovery processes. It enables to achieve excellent results in a short time, in the treatment of both acute and chronic diseases. A great benefit for both the patient and the physiotherapist.

[Find out more on page 46](#)

Laser therapy

Laser therapy is based on photochemical and photobiological effects in cells and tissues. Its application produces a number of responses from the body, including: increased blood flow, biostimulation, anti-inflammatory, anti-edematous and analgesic effects. This therapy is able to act on numerous acute and chronic conditions, relieving pain, reducing inflammation and accelerating recovery.

[Discover laser on page 48](#)

Professional ultrasound therapy

Ultrasound is ideal for treating muscular and nervous pathologies, for trauma recovery, in cases of both chronic and acute pathologies.

It is also indicated for aesthetic applications, such as cellulite treatment, tissue regeneration, vascularisation and lymphatic drainage.


What if an ultrasound device also had a dedicated electrotherapy module?

[Discover our products for ambulatory use on page 48](#)

REATHERM

Diathermy with the neuromodulation effect:
effective, safe, precise.

Optimum results from the very first session.
Whether for rehabilitation, sport, beauty.

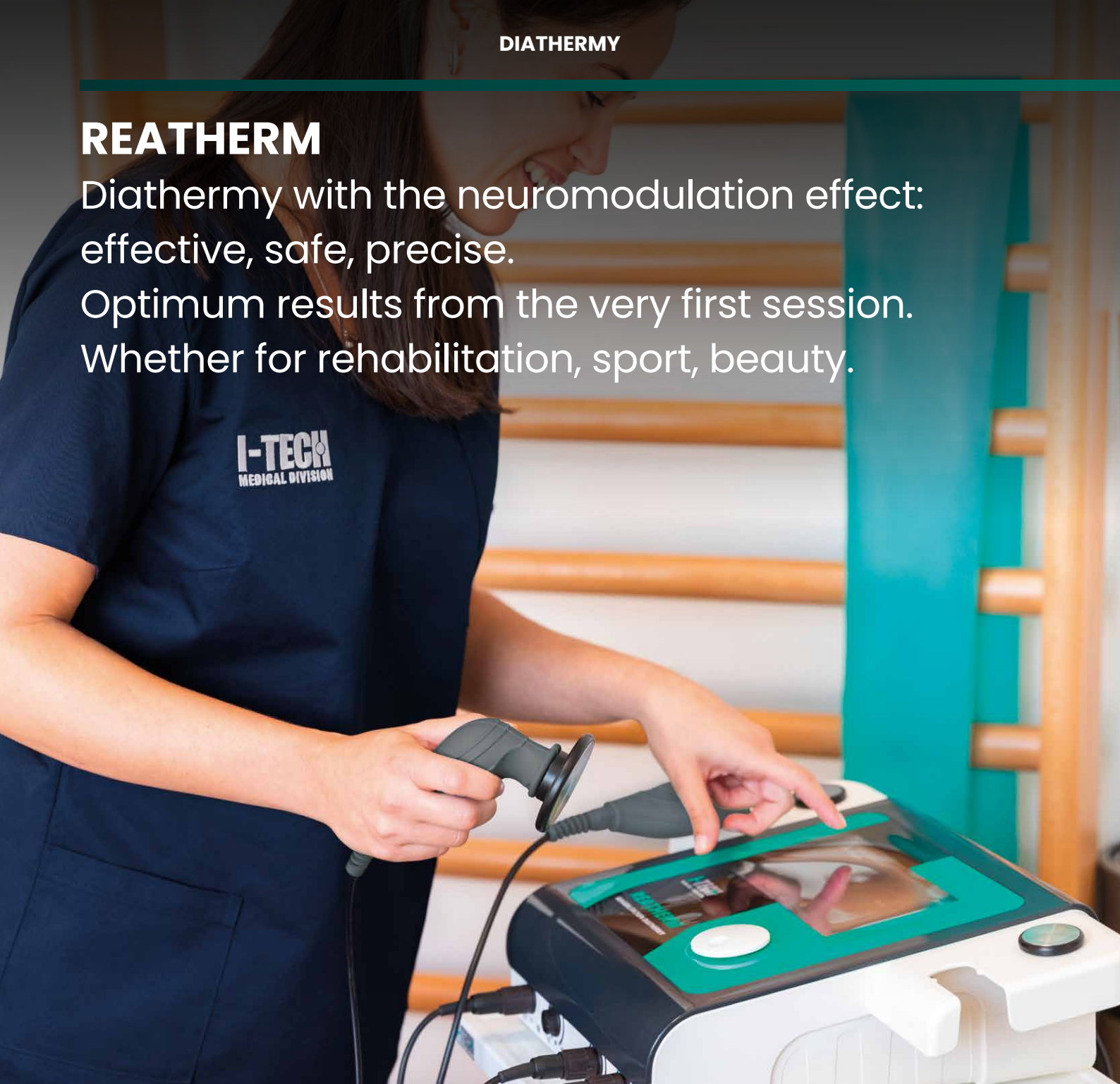

PROGRAMS

REHAB

01 Phantom limb	11 TMJ Dysfunction	21 ACL
02 Knee arthrosis	12 Sprain	22 Lymphedema
03 Arthrosynovitis	13 Muscle Distraction	23 Low back pain
04 Bursitis	14 Joint pain	24 Sciatica
05 Neck pain	15 Edema	25 Meniscopathy (meniscal synovitis)
06 Whiplash syndrome	16 Enthesitis	26 Diabetic foot
07 Chondropathy	17 Epicondylitis	27 Piriformis syndrome
08 Muscle contracture	18 Epitrochleitis	28 Muscle tear
09 Contusion	19 Disc hernia	29 Acute tendinitis
10 Muscular sprain	20 Scarring	30 Chronic tendinitis

MODULATION 1 ÷ 100 HZ

Comfortable treatment, also in athermia, neuromodulation effect. Not to be confused with the classic duty-cycle.

401 FREQUENCIES (300 ÷ 700 KHZ)

to adapt the treatment to the needs of the individual patient

10 FREE MEMORIES MODE CUSTOM

customised treatment, up to 60 minutes

UNIPOLAR AND BIPOLAR HANDPIECES

ergonomic

50 PRESET PROGRAMS

for a real guide to treatments

LIGHTWEIGHT, POWERFUL, TRANSPORTABLE

3 kg, 200 W

PROFESSIONAL TRAINING INCLUDED

Included



Add Troll-M!

3-shelf trolley with 4 swivel castors and 1 socket power socket.



SPORT

BEAUTY

31 Rotator cuff tendinopathy

01 Muscle contracture

01 Cellulite

32 Ulcer

02 Contusion

02 Collagen

33 Effusion

03 Muscle recovery

03 Edema

34 Arthroplasty (recovery)

04 Sprain

04 Draining massage

35 Fractures

05 Hematoma

05 Microlifting

06 Tendon injury

07 Superficial massage

08 Deep massage

09 Muscle sprain

10 Muscle tear

+ 10 FREE MEMORIES

LA500

Rapid intervention in the treatment of inflammation and degenerative diseases in the orthopaedic, neurological and dermatological fields.



CE 0068

I-TECH UT²

Double handpiece (5cm² and 1cm²) and choice of working frequency (1-3MHz) to adapt the treatment to the patient's therapeutic needs.



Included

Add Troll-M!

3-shelf trolley with 4 swivel castors and 1 socket power socket.



CE 0068

I-TECH UE

A complete device. Ultrasound therapy, electrotherapy with the possibility of combining the two therapies for even greater benefits.



Included

Add Troll-M!

3-shelf trolley with 4 swivel castors and 1 socket power socket.



CE 0068

Don't miss our content: useful tips for your therapy!

Sign up for our newsletter!



<https://itechmedicaldivision.com/en/newsletter/>



I-TECH
Life

Get the most
from your therapy.

**Download
our new app
and start your own
therapeutic journey.**



I-TECH

MEDICAL DIVISION

a **IACER** brand

I.A.C.E.R. Srl

via Enzo Ferrari 2

30037 Scorzè (VE) - Italia

Tel.: (+39) 041/5401356

itechmedicaldivision.com

C.F. e P.Iva: IT00185480274

iacer@iacer.it

