

USER MANUAL

MNPG370-03 Edition 31/01/2022

TAP2000

I.A.C.E.R. Srl

www.itechmedicaldivision.com

Introduction

The **TAP2000** device is a 6 solenoid applicator for low-frequency magnetotherapy. Designed to treat the spinal area by placing it on the back of a chair or armchair.

The device is classified as class I, in accordance with rule 1 (4.1) referred to in Annex VIII of Regulation (EU) 2017/745 of the European Parliament and of the Council, dated 5 April 2017, relating to medical devices.

Basic UDI-DI: 8019781PEMFLFTAPQS

UDI-DI: 08019781472090

*Technical
data*

Technical specifications:

- *Applicator size: 40x40cm;*
- *6 solenoids;*
- *2 connection cables;*
- *Compatible with MAG2000 models.*

Labelling





Label 1
(on the back of the device)

Label 1





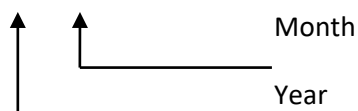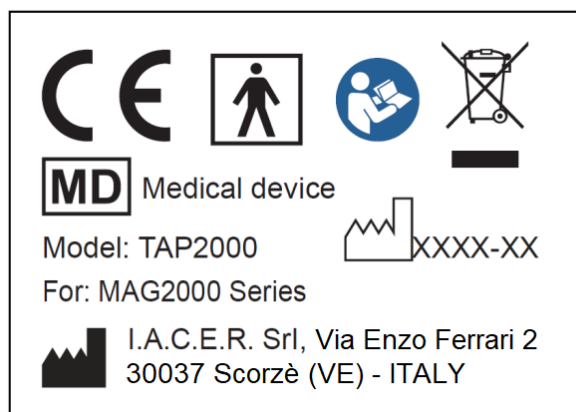



Cable label



Symbols:

Symbol	Meaning
	Manufacturer's logo.
	In compliance with regulation (EU) no. 2017/745
	Manufacturer data.
	Date of manufacture (YYYY-MM).

Symbol	Meaning
	Medical device
	Follow the instructions for use.
	WEEE directive for the disposal of electronic waste.
	Type BF applied part.
	Allowed temperatures (storage temperatures, on packaging).
	Relative humidity (relative humidity of storage, on packaging).

Transport and storage

There are no special care instructions for transport. For storage, the equipment is protected up to the following environmental conditions:

room temperature from +5 to + 40°C

relative humidity from 15 to 93%

Instructions for use

Instructions for using TAP2000

Follow the instructions in the User Manual supplied with the device in your possession. Connect the mat to the two channels of the device.


Follow the Instructions given in the user manuals of MAG2000 products.

Cleaning the TAP2000

To clean the TAP2000 it is recommended to disconnect the 6 solenoid applicator from the device before carrying out any operation. Clean the fabric with a cloth moistened with simple water and neutral soap and wait for it to dry completely before reusing the three solenoid applicator for a therapy session.

Information for disposal



The product is subject to the WEEE legislation (presence on the label of the symbol ) relating to separate waste collection: to dispose of the product, use special areas equipped for the collection of electronic material by contacting the competent authorities of your country or the manufacturer directly.

Maintenance and Support

The device does not require any particular routine maintenance. Please contact the manufacturer to provide for a check of the functionality of the device, every two years.

The manufacturer is the sole entity authorized to provide technical support for the TAP2000 applicator.

Should you need technical support, please contact:

I.A.C.E.R. S.r.l.

Via Enzo Ferrari, 2 • 30037 scorzè (VE)

Ph. 041.5401356 • Fax 041.5402684

Warranty

TAP2000 is covered by a 2-year warranty starting from the date of purchase. The warranty shall lapse in case of tampering, neglect and inappropriate use of the applicator and in case of any intervention carried out by personnel who is not authorized by the manufacturer or by the authorized dealer.

Incident reporting

In compliance with the provisions of Regulation (EU) 2017/745, the manufacturer places the need to report any serious accident in relation to the device to the attention of the user.

The report must be addressed:

- to the device manufacturer:

I.A.C.E.R. Srl

Via Enzo Ferrari 2 – 30037

Scorzè (VE) ITALY

Tel. +39 041 5401356 – Fax +39 041 5402684

e-mail: iacer@iacer.it



I.A.C.E.R. S.r.l.

Via Enzo Ferrari 2 - 30037, Scorzè (VE) – Italy
Tel.: (+39) 041 540 13 56 | Email: iacer@iacer.it

www.itechmedicaldivision.com

Share Capital: € 1.000.000 fully paid-up
Tax Code / VAT Number: IT 00185480274
Certified email: iacer@pec.it | SDI: SUBM70N