

USER MANUAL

MNPG379-03 Edition 04/07/2022

FTI1000

Band applicator

with radiant

antenna

I.A.C.E.R. Srl

www.itechmedicaldivision.com

Introduction

The **FTI1000** device is an applicator with a radiant antenna for high-frequency magnetotherapy.

It is easily adaptable to any area of the body thanks to the presence of the therapeutic elastic tape.

The device is classified as class I, in accordance with rule 1 (4.1) referred to Annex VIII of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5th April 2017 (and further modifications/integrations).

UDI-DI di base: 8019781PEMFHFASME

UDI-DI: 08019781113405

Technical data

Technical specifications:


- *Applicator size: 37x9 cm + elastic band;*
- *Cable length: 158 cm;*
- *Radiant antenna;*
- *1 connection cable;*
- *Compatible exclusively with MAG 1000 model.*

Labelling




Label 1

Label 1

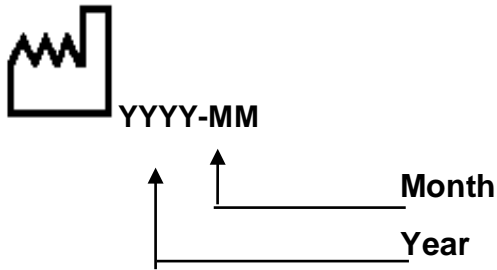



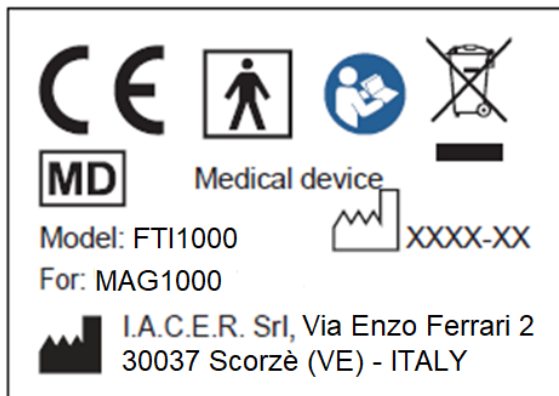


Cable label



Symbols:

Symbol	meaning
	Manufacturer's logo.
	In compliance with regulation (EU) n. 2017/745
	Manufacturer data.

Symbol	meaning
	Date of manufacture (AAAA-MM).
	Medical device.
	Follow the instructions for use.
	WEEE directive for the disposal of electronic waste.
	Type BF applied part.

Transport and Storage

There are no special care instructions for transport. For storage, the equipment is protected up to the following environmental conditions:

room temperature from +5 to + 40°C

relative humidity from 30 to 80%

Instructions for use

Instructions for using FTI1000

Follow the instructions in the User Manual supplied with the device in your possession.

Follow the Instructions given in the user manuals of the MAG1000.


How to take care of FTI1000

Cleaning the FTI1000

To clean the FTI1000 device, it is recommended to disconnect the applicator from the device before performing any operation. Clean the fabric with a cloth moistened with a solution of water and alcohol (20% solution) and wait for it to dry completely before reusing the applicator for a therapy session.

Information for disposal



The product is subject to the WEEE legislation (presence on the label of the symbol ) relating to separate waste collection: to dispose of the product, use special areas equipped for the collection of electronic material by contacting the competent authorities of your country or the manufacturer directly.

Maintenance and Support

The device does not require any particular routine maintenance. Please contact the manufacturer to provide for a check of the functionality of the device, every two years. The manufacturer is the sole entity authorized to provide technical support for the FTI1000 applicator. Should you need technical support, please contact:

I.A.C.E.R. S.r.l.

Via Enzo Ferrari, 2 • 30037 Scorzè (VE)

Tel. 041.5401356 • Fax 041.5402684

Garanzia

FTI1000 is covered by a 2-year warranty starting from the date of purchase. The warranty shall lapse in case of tampering, neglect and inappropriate use of the applicator and in case of any intervention carried out by personnel who is not authorized by the manufacturer or by the authorized dealer.

Incident reporting

In compliance with the provisions of Regulation (EU) 2017/745, the manufacturer places the need to report any serious accident in relation to the device to the attention of the user.

The report must be addressed:

- to the device manufacturer:

I.A.C.E.R. Srl

Via Enzo Ferrari 2 – 30037

Scorzè (VE) ITALY

Tel. +39 041 5401356 – Fax +39 041 5402684

e-mail: iacer@iacer.it



I.A.C.E.R. S.r.l.

Via Enzo Ferrari 2 - 30037, Scorzè (VE) – Italy

Tel.: (+39) 041 540 13 56 | Email: iacer@iacer.it

www.itechmedicaldivision.com

Share Capital: € 1.000.000 fully paid-up

Tax Code / VAT Number: IT 00185480274

Certified email: iacer@pec.it | SDI: SUBM70N